EN CueSee® Glucose EQAS

Intended Use

CueSee® Glucose EQAS is an unassayed glucose quality control material especially designed for application in External Quality Assessment and Proficiency Testing Schemes to verify the precision and accuracy of blood glucose meters (BGMs).

Reagents

CueSee® Glucose EQAS contains bovine packed blood cells and plasma and is available at various physiologically relevant levels.

Storage and stability

CueSee® Glucose EQAS must be stored at 2-8 °C (35-46 °F). Stored unopened at this temperature it is guaranteed stable until the expiration date as indicated on the label on the box. After reconstitution CueSee® Glucose EQAS is stable for 4 hours at 15-30 °C (59-86 °F).

Procedure

1. Read all instructions carefully before preparing the sample. See instructions for preparing the ACU-Drop® II sample. Visit www.eurotrol.com for an instructional video for preparing an ACU-Drop II sample.
2. Allow the product to stand for 15 minutes at room temperature (15 – 30°C / 59 – 86°F). Note: Do not put vials back into refrigerator once exposed to room temperature.
3. Follow instructions on the back side of this document carefully to obtain a homogeneous CueSee® Glucose EQAS sample. **NOTE: to avoid excessive foaming make sure that every step is performed slowly and gently.**
4. The sample may be used at 15 - 30 °C (59 - 86 °F) for 4 hours after reconstitution.

Precautions

1. For in vitro diagnostic use only.
2. This is a bovine animal product and does not contain biohazards to humans such as HepB, HepC and HIV. This product is free from TSE.
3. This product should not be disposed of in general waste. Consult local environmental authorities for proper disposal.

Expected results

Each lab should establish its own results and report them at CueSee® Online (<http://www.cuesee-online.com>) to obtain peer group analysis and summary reports.

For further information or technical assistance, please contact:

Eurotrol B.V.

Phone +31 318 695 777
 Fax +31 318 695 770
 E-mail office@eurotrol.com
 Website www.eurotrol.com

 Eurotrol B.V., Keplerlaan 20, 6716 BS Ede, The Netherlands

Symbols used

 Attention, see instructions for use	 Batch number
 Use by	 Temperature limitation
 Manufacturer	 Reference number

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INP 254-FR-R01 20150903

EN Instructions for preparing an ACU-Drop® II sample

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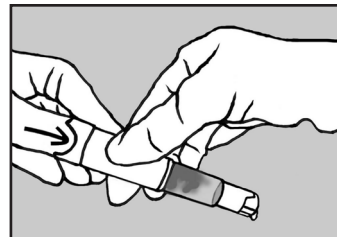
EN Instruction A - Sampling with Dropper Tip

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
EN A1 - Press the button firmly until it is completely pushed in and level with the indentation.

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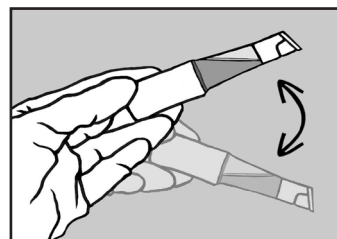
EN A2 - Gently rock and swirl at least 5 times to mix the liquids completely.

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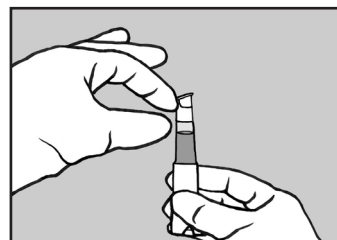
EN A3 - Hold the ACU-Drop II with cap facing up and flick the cap firmly several times to make sure that no liquid remains inside the dropper tip.

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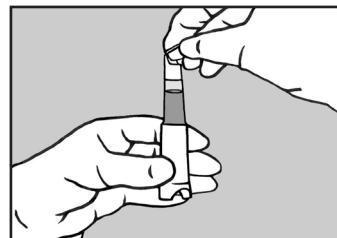
EN A4 - Open the flip cap very carefully by lifting the cap just a little bit to release the pressure. - Only with first time use -

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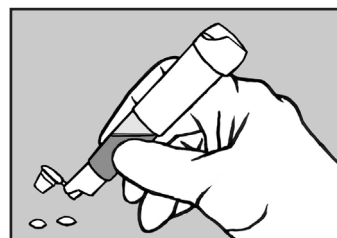
EN A5 - The first 2 drops should be dispensed and discarded.

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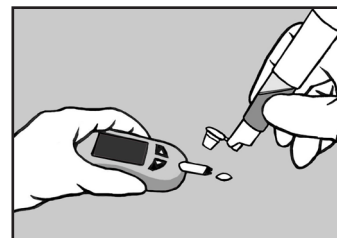
EN A6 - The ACU-Drop II is now ready to use to apply a sample drop for testing.

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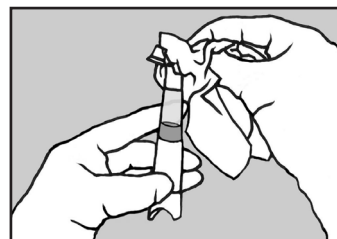EN A7 - After each use, be sure to wipe the dropper tip and inside of cap to keep them clean.

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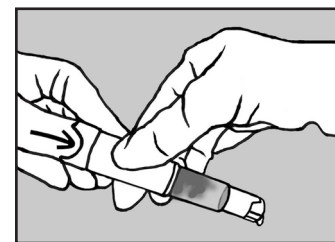
EN Instruction B - Sampling with Syringe

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
EN B1 - Press the button firmly until it is completely pushed in and level with the indentation.

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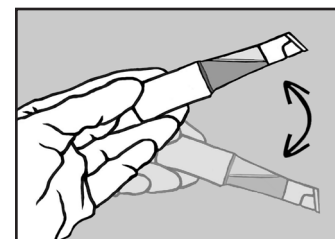
EN B2 - Gently rock and swirl at least 5 times to mix the liquids completely.

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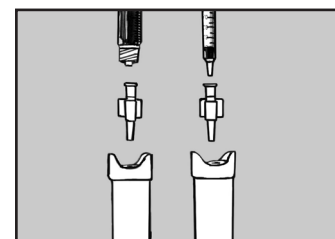
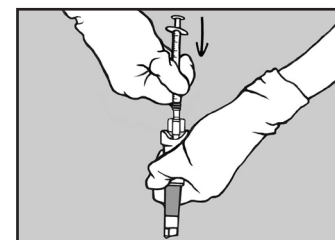
EN B4 - Hold the ACU-Drop II with cap facing down and flick the bottom firmly several times to be sure that all the liquid settles. Push the adaptor tip attached to the syringe firmly into the luer fitting on the button of the ACU-Drop II.

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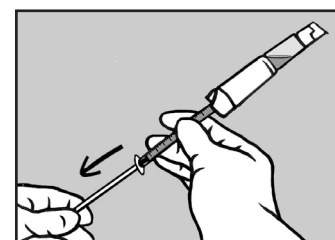
EN B5 - Hold the ACU-Drop II with cap facing up and, using both hands on the syringe, withdraw the contents.

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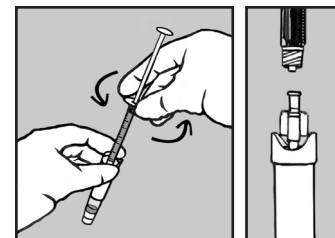
EN B6 - Hold the ACU-Drop II with cap facing down and carefully unscrew the syringe from the adaptor, leaving the adaptor in the button of the ACU-Drop II.

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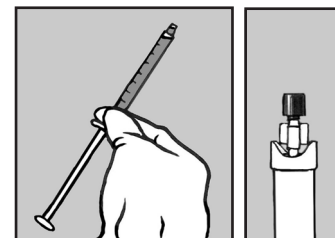EN B7 - Seal the adaptor with screw cap to discard or re-use the ACU-Drop II. (Possibility for re-use: see specific product instructions for shelf-life after reconstitution.) Use the syringe to deliver your sample.

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